

SCHIFF'S CHAIR CLEANER & POLISH may be used on non-porous environmental surfaces and smooth upholstery for a streak free shine. Its formulation is gentle and safe for use on leather & vinyl upholsteries, stainless steel appliances and fixtures, Formica or stone-type solid counters and tables, electronics, plastics and more.

SCHIFF'S CHAIR CLEANER & POLISH does not contain mineral oil, is low in VOCs and its non-flammable formula helps to prevent static build-up.

Directions:

Remove any buildup of dirt and grime. Spray **SCHIFF'S CHAIR CLEANER & POLISH** across the surface. With a soft cloth or paper towel, wipe the surface until dry. Repeat cleaning as needed to maintain a clean, streak-free, polished surface.

maxill inc.

St. Thomas, ON, Canada N5R 6C8
1-800-268-8633 www.maxill.com

Made in Canada with domestic and imported parts

maxill

ITEM 60020

SCHIFF'S

CHAIR CLEANER & POLISH

SPECIALLY FORMULATED FOR USE TOGETHER WITH



DENTAL CHAIRS • LAB EQUIPMENT • WORK SURFACES

BUFFS TO A
**STREAK-FREE
SHINE**



Warning
Causes eye irritation

Prevention:

Wash from skin thoroughly after handling.

Response:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do so. Continue rinsing.

If eye irritation persists: Get medical advice/attention.

Storage:

Protect from freezing or cold.
Store above 4°C at room temperature.

Cleaning Care for Dental Chairs

In addition to disinfecting, a cleaning protocol is required by the chair manufacturer. To learn more about how **SCHIFF'S CHAIR CLEANER & POLISH** can be used to help prolong the life of your dental chair upholstery, call 1-800-268-8633 or visit

www.maxill.com/ca/chaircare

REV.2018/09



946 mL

0 60705 60020 2